

DOM'HOUS

Protection anti-acariens

Housses de protection anti-acariens pour les personnes allergiques

UNE LARGE GAMME À VOTRE DISPOSITION * :

MATELAS	COUETTES
ENFANT	140 x 200
60 x 120 x 15	200 x 200
60 x 140 x 15	220 x 240
1 PERSONNE	240 x 260
90 x 190 x 19	
90 x 190 x 23	OREILLERS
90 x 200 x 19	65 x 65
90 x 200 x 23	60 x 70
100 x 200 x 23	
120 x 200 x 23	TRAVERSINS
2 PERSONNES	90
140 x 190 x 19	140
140 x 190 x 23	160
140 x 200 x 15	
160 x 200 x 19	
160 x 200 x 23	
180 x 200 x 15	
200 x 200 x 23	

* Possibilité de commander des tailles spécifiques sur simple demande

Les housses Dom'HouS ont été conçues pour des personnes allergiques aux acariens, la literie étant un lieu de prédilection de leur développement.

Ce sont des enveloppes complètes recouvrant l'ensemble de votre literie, confectionnées dans un matériau de haute qualité, totalement étanche aux acariens, et perméable à la vapeur d'eau. Leur utilisation vous assure nuit après nuit, protection et confort.

Pour une efficacité maximum, tous les éléments de votre literie doivent être protégés et les housses doivent être maintenues en place pendant au moins 3 mois.

En effet, une nette diminution des symptômes allergiques nocturnes et matinaux est observée lorsque le dormeur est totalement isolé des acariens.

Dom'HouS est le traitement naturel de l'allergie aux acariens.

L'ALLERGIE AUX ACARIENS :

- est responsable d'environ 60 % des allergies respiratoires ;
- est provoquée par les acariens mais aussi par leurs déjections, reconnues comme particules allergisantes;
- est majorée au contact des lieux d'habitats privilégiés des acariens (matelas, oreillers, moquettes...).

MODE D'EMPLOI :

- Mettre à nu votre matelas, oreiller et couette ;
- Aspirer toutes les faces de votre matelas ;
- Installer votre housse Dom'HouS ;
- Remettre votre literie habituelle : drap, housse de couette, taie d'oreiller.

PRÉCAUTIONS IMPORTANTES :

- Ne pas utiliser les housses sur une literie pliable (risque de déchirure) ;
- Les housses doivent être rangées la fermeture éclair fermée et tenues hors de portée des enfants pendant les périodes de non utilisation ;
- Stocker les housses dans un endroit propre et sec ;
- Ne pas laisser à la portée des enfants. Sac plastique (risque de suffocation).

ENTRETIEN :

Votre housse se lave et s'entretient à raison de 2 lavages par an à 60°C :



COMPOSITION ET PERFORMANCES :

- Microfibre 100% polyester ;
- Poids : 89 g/m² ;
- Résistance garantie au lavage à 60°C ;
- Perméabilité : 7 m²Pa/W (ou Unité RET).

GARANTIES :

- Votre housse Dom'HouS est garantie 10 ans dans la mesure où vous respectez strictement les indications relatives à son entretien figurant dans cette notice ;
- En cas d'éventuels vices de fabrication, STALLERGENES s'engage à remplacer sans frais le produit défectueux ;
- Toutes nos housses Dom'HouS sont conformes à la directive 93/42/CEE relatives aux dispositifs médicaux.



Année d'apposition du marquage CE : 1998

TESTS D'EFFICACITÉ ET D'INNOCUITÉ :

- Innocuité : **Oeko-Tex®**
- Efficacité : **effet barrière contre les acariens** test effectué par l'institut Hohenstein

Mise à jour de la notice : mars 2015




KARA INDUSTRIE

Rue Sidi Nasr

BP 143 - 5012 SAHLINE - TUNISIE

Tél. : +216 73 52 79 00 - Fax : +216 73 52 79 01

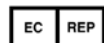

KARAPHARM

Village entreprise Saint Henri - Lot 311

Rue Anne Gacon - 13 016 MARSEILLE - FRANCE

Tél. : + 33 4 91 57 05 57 - Fax : + 33 4 91 57 03 90

Distribué par



WWW.stallergenes.com

DOM'HOUS

Anti-mite covers

Anti-mite covers for allergic people

A LARGE RANGE AT YOUR DISPOSAL* :

MATTRESS	DUVETS
CHILDREN	140 x 200
60 x 120 x 15	200 x 200
60 x 140 x 15	220 x 240
1 PERSON	240 x 260
90 x 190 x 19	
90 x 190 x 23	PILLOWS
90 x 200 x 19	65 x 65
90 x 200 x 23	60 x 70
100 x 200 x 23	
120 x 200 x 23	BLOSTERS
2 PERSONS	90
140 x 190 x 19	140
140 x 190 x 23	160
140 x 200 x 15	
160 x 200 x 19	
160 x 200 x 23	
180 x 200 x 15	
200 x 200 x 23	

* Special sizes possible on command.

Dom'HouS covers have been specially designed for people who are allergic to mites, bedding being the preferred habitat for mites as it provides all the conditions most propitious to their growth.

Dom'HouS are complete covers that envelop the whole bedding. They are made of a high-tech fabric which perfectly isolates the body from mites and is permeable to water vapour. Every night, their utilisation ensures protection and comfort.

For a maximum efficacy, all the bedding elements must be protected and covers must be maintained for at least 3 months. It has been observed that the consistent use of antimite covers results in mitigation of nocturnal and morning symptoms of allergy to mites.

Dom'HouS is the natural treatment of allergy to mites.

MITE ALLERGY:

- accounts for about 60% of respiratory allergies;
- is triggered by mites but also by their faeces which are recognised as allergenic particles;
- is increased in mites' favourite habitat such as mattress, duvet, pillow...

INSTRUCTIONS FOR USE:

- Take off the bedlinen from your mattress, pillow, duvet or bolster;
- Vacuum clean all the sides of your bedding;
- Place your Dom'HouS cover;
- Recover with your usual bedlinen.

IMPORTANT PRECAUTIONS:

- Do not use the cover on a foldable bed (risk of tearing);
- When stored the cover should be zipped;

Dom'Hous Europe_NOT003-15

- Store the cover in a clean and dry place;
- Keep out of reach of children. Plastic bag may cause suffocation.

MAINTENANCE:

Wash your cover at 60°C twice a year.



COMPOSITION AND PERFORMANCE:

- 100% polyester microfiber;
- Weight: 89 g/m²;
- Washing guaranteed at 60°C;
- Permeability: 7 m²Pa/W (or RET Unit).

GUARANTEES:

- Your Dom'Hous cover is guaranteed for 10 years (provided instructions for use have been strictly respected);
- In the event of defective materials or manufacturing, the cover will be replaced free of charge;
- Dom'Hous covers comply with Directive 93/42/EEC concerning medical devices.



Initial EC Marking Certification : 1998

SAFETY AND PERFORMANCE TEST RESULTS:

- Safety: **Oeko-Tex**[®]
- Performance: **Barrier effect towards mites tested by the Hohenstein Institute**

Date of revision of the leaflet: March 2015




KARA INDUSTRIE

Rue Sidi Nasr

BP 143 - 5012 SAHLINE - TUNISIE

Tel.: +216 73 52 79 00 - Fax: +216 73 52 79 01

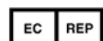

KARAPHARM

Village entreprise Saint Henri - Lot 311

Rue Anne Gacon - 13 016 MARSEILLE - FRANCE

Tel.: + 33 4 91 57 05 57 - Fax: + 33 4 91 57 03 90

Distributed by



WWW.stallergenes.com